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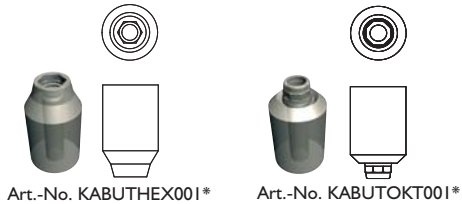
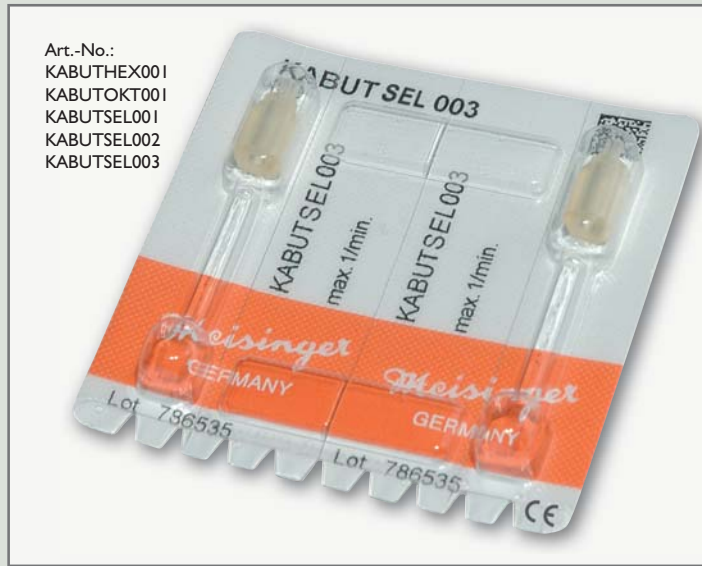
- Innovative and particularly simple handling of implant based prosthetics
- Time-saving processes combined with extremely high precision and optimum fit
- Cost-effective procedure due to a reduction in components
- Irritation free gingiva
- High-precision special-purpose plastic, temperature stability up to +135°C (275°F), can be sterilised and is opaque to radiological imaging

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CE

* With the reuse of disposable products the risk of infection cannot be excluded and a risk-free functional safety cannot be guaranteed

Bone Management® is a registered trademark of the Hager & Meisinger GmbH, Germany

Easy-Handling-Abutments

Easy Handling Prosthetic Abutments

85FL056-0309

Hager & Meisinger GmbH

Hansemannstr. 10
41468 Neuss • Germany
Tel.: +49 (0)21 31-20 120
Fax: +49 (0)21 31-20 12.222
Internet: www.meisinger.de
www.bone-management.de
E-mail: info@meisinger.de

Meisinger USA, L.L.C.

7442 South Tucson Way, Suite 130
Centennial, Colorado 80112 • USA
Tel.: +1 (303) 268-5400
Toll free: +1 (866) 634-7464
Fax: +1 (303) 268-5407
Internet: www.meisingerusa.com
www.bone-management.com
E-mail: info@meisingerusa.com

MEISINGER
GERMANY / USA

Easy-Handling-Abutments

developed with Dr. Althoff, Dr. Ortner

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Easy-Handling-Abutments consist of high-precision, certified plastic which has been in use in other fields of medicine for several decades. It can be trimmed in situ or processed as required in the laboratory. This patented process enables osseointegrated superstructures to be made quickly and easily (direct moulding of the plastic using the lost-plastic moulding technique) while ensuring high dimensional accuracy. This provides a prosthetic concept which, due to its simple handling, opens up new opportunities for both experienced practitioners and those at an early stage in their career.

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Once osseointegration has been completed and the implant surface uncovered, the Easy-Handling-Abutment is initially fixed by screwing it in place in the mouth using the occlusal screw (with the required torque). If required, radiological imaging can be used to verify that the abutment is correctly positioned gap-free on the surface of the implant.

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The abutment can be ground to fit and, in particular, aesthetically shaped to produce the desired emergence profile. Ground-in longitudinal grooves facilitate repositioning in the subsequent impression. The abutment is then unscrewed, attached to the corresponding model analogue and repositioned. The cast is then made using normal laboratory procedures. Any required reshaping is easily done.

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The casting process is the same as for wax models. Modifications, such as shaping in the laboratory are possible and will reduce chair time. Once the casting and fitting have been completed, the secondary construction (e.g. a full-veneer metal based ceramic crown) can be directly manufactured. Both parts can be brought together in the final session, in other words the inner construction is screwed in place as normal, verified with radiological imaging, after which the superstructure is cemented in place.

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compatible with the Branemark MK II implant system, diameters 3.75 mm and 4.00 mm, and with the Dental Ratio Hexatec® implant system, diameter 3.75 mm

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compatible with the Straumann Standard and Standard Plus implant systems, diameters 3.30 mm, 4.10 mm, 4.80 mm with the regular neck 4.8 mm, and with the Dental Ratio Oktagon® implant system, diameters 4.10 mm and 4.80 mm

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General instructions:

- All products delivered are unsterile therefore, before initial and each further potential application to be treated (cleaning / disinfection / sterilisation).

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