
	Vorkörner Initial bur Fresa inicial	Allportbohrer Allport bur Fresa Allport	Lagerfräser Ablative burs Fresas ablativas	Spiralbohrer Twist drills Taladros espirales	Hülseaufnahme Sleeve grip Detentor de portabrocas			
Fig.	186RF	141RF	TC21X	TC21X	203RF	203RF	CHA38	CHA60
Shank <sup>1</sup>	204	204	205	205	205	205		
Size <sup>2</sup>	018	031	040	060	038	060		
Length mm	12,0	0,3	14,0	14,0	14,0	14,0		
Internal diameter							3,8	6,0
External diameter								

<sup>1</sup> 204=RA, 205=RA L <sup>2</sup> Largest working part diameter in 1/10 mm

	Bohrhülsen Drill sleeves Casquillos portabrocas											
Fig. C000B...	I3H38	I3H60	I2H38	I2H60	I1H38	I1H60	I0H38	I0H60	9H38	9H60	8H38	8H60
Length mm	13,0	13,0	12,0	12,0	11,0	11,0	10,0	10,0	9,0	9,0	8,0	8,0
Drilling depth	1,0	1,0	2,0	2,0	3,0	3,0	4,0	4,0	5,0	5,0	6,0	6,0
Internal diameter	3,8	6,0	3,8	6,0	3,8	6,0	3,8	6,0	3,8	6,0	3,8	6,0
External diameter	5,35	7,55	5,35	7,55	5,35	7,55	5,35	7,55	5,35	7,55	5,35	7,55

Internal diameter External diameter

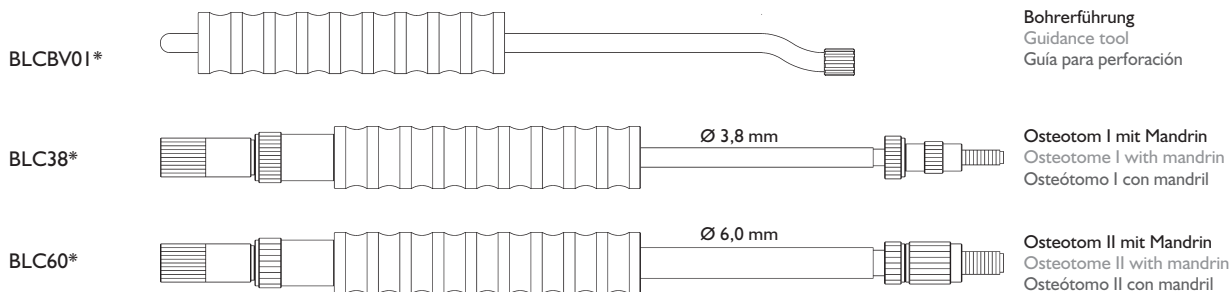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- Minimally invasive lifting of internal sinus
- Optimal use of existing bone substance independent of vertical height
- Safe, defined and simple access to the maxillary sinus mucosa
- Complication free lifting by up to approx. 10 mm through extended balloon
- Suitable to prepare the jaw bone for the implantation of all current systems
- Possibility of direct implantation by the application of the Osteotome I with the diameter 3.80 mm

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Bone Management® is a registered trademark of the Hager & Meisinger GmbH, Germany

Der Ballonkatheter ist separat unter der Art.-No. BLC0K\* erhältlich  
The balloon expander can be ordered separately under the following Art.-No. BLC0K\*  
El catéter con balón está disponible por separado bajo el Art.No. BLC0K\*  
steril / sterile / esterilizar

\* With the reuse of disposable products the risk of infection cannot be excluded and a risk-free functional safety cannot be guaranteed

## Balloon-Lift-Control

### Balloon-Assisted Sinus Floor Elevation

85FL045-0309

#### Hager & Meisinger GmbH

Hansemannstr. 10  
41468 Neuss • Germany  
Tel.: +49 (0) 21 31-20 12 0  
Fax: +49 (0) 21 31-20 12 222  
Internet: www.meisinger.de  
www.bone-management.de  
E-mail: info@meisinger.de

#### Meisinger USA, L.L.C.

7442 South Tucson Way, Suite 130  
Centennial, Colorado 80112 • USA  
Tel.: +1 (303) 268-5400  
Toll free: +1 (866) 634-7464  
Fax: +1 (303) 268-5407  
Internet: www.meisingerusa.com  
www.bone-management.com  
E-mail: info@meisingerusa.com

MEISINGER  
GERMANY / USA

## Balloon-Lift-Control

developed with Prof. Dr. Benner, Dr. Bauer

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Balloon-Lift-Control (BLC) is a very safe procedure used for the pre-augmentative stripping of the maxillary sinus mucosa. It is minimally invasive. Independently of the vertical height of the osseous sinus floor, the mucosa can be lifted by more than 10 mm. BLC consists of two osteotomes with the help of a balloon expander that is inserted into the maxillary sinus after impression of the weakened floor. Subsequently the balloon is extended to the required size using radio-opaque liquid or sterile saline solution. The necessary volume equals the volume of augmentation required.

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The use of the initial bur combined with the respective shell prevents the special bur from slipping. If required, the allport bur or one of the ablative burs can refresh and smooth the bone.

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Depending on the existing bone and the size of the implant to be inserted choose the guidance shell, the special bur, and the osteotome with the larger (6.0 mm) or smaller (3.8 mm) diameter. Now the appropriate guidance shell is fastened onto the guidance tool, which is then held in front of the prepared alveolar crest.

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Subsequently, and in order to thin the sinus floor you use the special bur to drill through the guidance shell into the alveolar crest until the thickness of the remaining bone measures 1 mm. The perfectly matching tools ensure that

this pre-determined height of the remaining bone is maintained.

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and the osteotome with the larger (6.0 mm) or smaller (3.8 mm) diameter. Now the appropriate guidance shell is fastened onto the guidance tool, which is then held in front of the prepared alveolar crest.

The tip of the osteotome, with the help of the laser markings (mm scale), is aligned and set to the appropriate drill canal depth then fixed with the wider counter screw. The exit of the inner mandrin is, with help of the lower knurl-screw, adapted to the remaining bone-height accordingly. This is accomplished with the aid of the laser markings (mm scale) on the lower part of the mandrin with the subsequent fixation of the lower screw. With this one can precisely set the withdrawal point of the mandrin. Subsequently the tip of the osteotome is positioned in the pre-formed canal the is then initiated.

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By slightly tapping on the mandrin grip, the compacta on the maxillary sinus side is fractured in a controlled fashion. Prior to this, the part of the mandrin that is now rising above the top of the shaft was accurately

adjusted to the height of the remaining bone.

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Remove of the mandrin from the osteotome. Prior to inserting the balloon expander, the air must be evacuated i.e. it must be pre-filled with fluid (see the detailed description of the balloon expander), then it is inserted into the guiding canal of the osteotome as far as it will go. In doing so, the blue extension at the end of the catheter shaft is pushed right down into the deepening of the osteotome grip.

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This is followed by a repeated (carried out at least 5 times) slow and controlled extension of the balloon with a maximum of approximately 3 ml of fluid (Ultravist® 240 or physiological, 0.9 % NaCl solution). This procedure ensures the stripping and lifting of the maxillary sinus mucosa. If required, a respective X-ray image can be generated with the extended balloon positioned in situ.

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At the end, the balloon is emptied again and removed with the osteotome. Fill the area with the predetermined bone grafting material. By the application of the osteotomes with the diameter of 3.8 mm, a direct implantation is possible.

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#### General instructions:

- All products delivered are unsterile therefore, before initial and each further potential application to be treated (cleaning / disinfection / sterilisation).

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