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- Safe and simple extraction of roots
- Maximal protection of the tooth socket
- Possibility of drilling even if minute root fragments or foreign bodies are present in the root
- Possibility of drilling independently of anatomic root canal

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Bone Management® is a registered trademark of the Hager & Meisinger GmbH, Germany

Benex®-Control

Root Extraction System

	Spiralbohrer Twist drills Taladros espiral (2 St./2 p./2 pzs.)	Extraktionsschrauben Extraction screws Tornillos de extracción (2 St./2 p./2 pzs.)	Schraubendreher Extraction screw key Atronilladores		
CE 0044					
Fig. BE001230...	...030	...060	...070	...040	...050
Shank ¹	206				
Length mm	17,0	10,0	16,0	7,0	15,0
	1,3	1,3	1,3		
	1,8	1,6	1,6	10,0	10,0

¹ 206=RA XL Internal diameter External diameter

	Osteotomieinstrumente Osteotomy instruments Fresas para osteotomia			
Fig. BE001230...	HM33IL	859L	859	847
Shank ¹	205	204	204	204
Size ²	010	010	018	018
Length mm	5,5	12,0	10,0	8,0

¹ 204=RA, 205=RA L ² Largest working part diameter in 1/10 mm

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Benex®-Control (Root Extraction System) is ideally suited for the extraction of roots and root fragments. Due to the innovative construction of the extractor, the root can be removed very easily and in an extremely controlled manner without causing injury to the bone. When dealing with multi-rooted teeth it is essential to separate the roots before attempting extraction.

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Severing the Sharpey fibres (desmodontal fibres) in the sulcus using a periostome. Long, powerful roots have to be gently luxated using a fine lever or periostome for 30 seconds in an axial direction, ensuring that no transverse motion is applied.

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Initially, a diamond coated twist drill or standard twist drill is used to perform an under-dimensioned drilling operation on the axis and the centre of the root or the remaining root fragment. The drill should reach a depth of approximately 7 mm in the firm hard tissue as drilling deeper is unnecessary. Ideally, this action should be carried using a water cooling system. In order to be better able to remove the drilling debris, the twist drill or pilot bur should be used in a vertical up and down motion.

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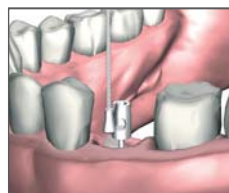
Choose the correct short or long extraction screw for access to the root to be extracted. Carefully screw the extraction screw securely into the root that has been prepared through drilling. Use the appropriate extraction screw key for access.

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
The extractor will be positioned on top of the adjoining teeth, which are protected by the Teflon® bite plate. The opening of the round, rotatable bite plate will be directed towards the vestibule, which should ensure an unobstructed view of the extraction screw. Subsequently the traction string will be secured into the extraction screw, guided over the return pulley

and hung onto the hook of the extraction slide while the screw and the string should be pointing in the same axial direction.

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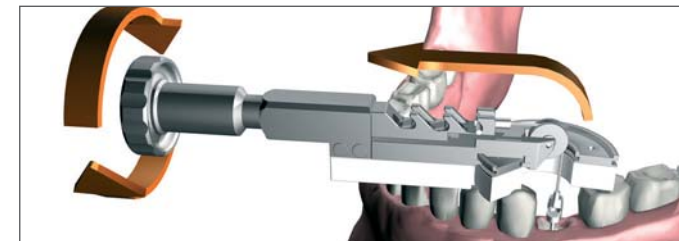Once the extractor has been correctly positioned, extraction takes place by means of slow, controlled turning of the handscrew. If the roots are long and powerful, the periodontal fibres should be pretensioned at intervals using periostomes.

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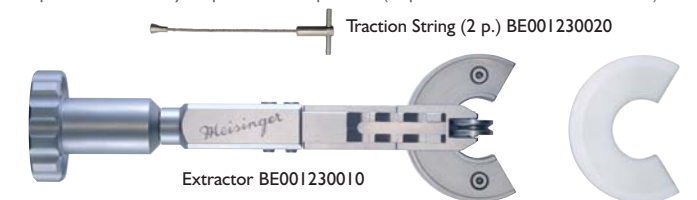
General instructions:

- All products delivered are unsterile therefore, before initial and each further potential application to be treated (cleaning / disinfection / sterilisation).

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Maßstab 1:2 / illustrated 1:2 / gráfica en escala 1:2

Teflon® is a registered trademark of DuPont, USA