# Benex®-Control Professional

## Root Extraction System

CE 0044	Diamantierte Spiralbohrer Diamond-coated twist drills Taladros espiral diamantada		Extraktionsschrauben Extraction screws Tornillos de extracción				Schraubendreher Extraction screw key Atornilladores		Pilotbohrer Pilot burs <sup>3</sup> Fresas piloto		
	Fig. BE001230...	030	035	060	070	080	085	040	050	A2001	B2001
Shank <sup>1</sup>	206	206								204	205
Size <sup>2</sup>										013	016
Length mm	17,0	17,0	10,0	16,0	10,0	16,0				12,0	12,0
□	1,3	1,8	1,3	1,3	1,8	1,8					
□	1,8	2,3	1,6	1,6	2,1	2,1	10,0	10,0	1,3	1,6	

<sup>1</sup> 204=RA, 206=RA XL, 314=FG, 316=FG XL, 317=30 mm    <sup>2</sup>Largest working part diameter    <sup>3</sup>Alternative to diamond-coated twist drills

□ Internal diameter    □ External diameter

Bone Management® is a registered trademark of the Hager & Meisinger GmbH, Germany

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- Safe and simple extraction of roots
- Maximum protection of the tooth socket
- Possibility of drilling even if minute root fragments or foreign bodies are present in the root
- Possibility of drilling independently of anatomic root canal

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Benex®-Control Professional (Root Extraction System) is ideally suited for the extraction of roots and root fragments. Due to the innovative construction of the extractor, the root can be removed very easily and in an extremely controlled manner without causing injury to the bone. When dealing with multi-rooted teeth it is essential to separate the roots before attempting extraction.

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Severing the Sharpey fibres (desmodontal fibres) in the sulcus using a periotome. Long, powerful roots have to be gently luxated using a fine lever or periotome for 30 seconds in an axial direction, ensuring that no transverse motion is applied.

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Initially, a diamond coated twist drill or standard twist drill is used to perform an under-dimensioned drilling operation on the axis and the centre of the root or the remaining root fragment. The drill should reach a depth of approximately 7 mm in the firm hard tissue as drilling deeper is unnecessary. If up until now only the pilot drill was used, we recommend an additional widening of the cavity with the diamond spiral drill which will subsequently make it easier to screw in the extraction screws. Ideally, this action should be carried using a water cooling system. In order to be better able to remove the drilling debris, the twist drill or pilot bur should be used in a vertical up and down motion.

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
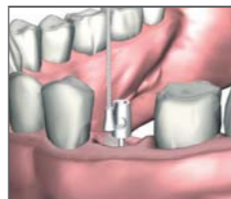
Choose the correct short or long extraction screw for access to the root to be extracted. Carefully screw the extraction screw securely into the root that has been prepared through drilling. Use the appropriate extraction screw key for access.

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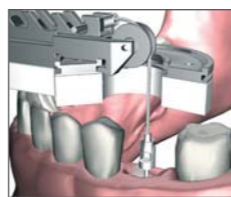
The extractor will be positioned on top of the adjoining teeth, which are protected by the Teflon® bite plate. The opening of the round, rotatable bite plate will be directed towards the vestibule, which should ensure an unobstructed view of the extraction screw. Subsequently the traction string will be secured into the

extraction screw, guided over the return pulley and hung onto the hook of the extraction slide while the screw and the string should be pointing in the same axial direction.

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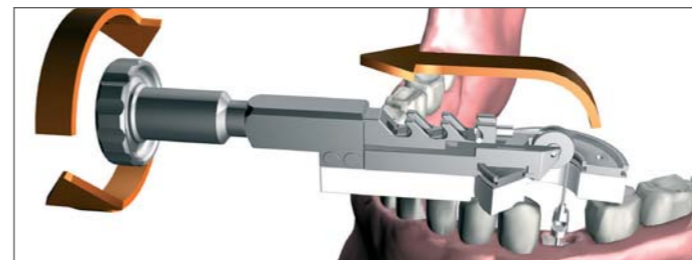Once the extractor has been correctly positioned, extraction takes place by means of slow, controlled turning of the handscrew. If the roots are long and powerful, the periodontal fibres should be pretensioned at intervals using periotomes.

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The best possible vertical transmission of power requires the existence of a right-angled plane to rest the extractor on. The newly developed quadrant support offers such a right-angled plane for positioning the extractor. This quadrant allows for uneven or open ended situations to be corrected. Primarily, the use of a quadrant support is required for extractions in the lower jaw area as well as for extracting palatal roots in the upper jaw.

- 1.) Fill the quadrant support with a rapidly hardening impression material and mould a recess in the area of the buccal opening.
- 2.) Position the quadrant support with opening in the area of the root to be extracted, at right angles to the root axis.
- 3.) Remove the quadrant support when the impression materials has begun to harden. Full hardening takes place extraorally.
- 4.) Luxate the root. Subsequently, use the Benex screw and reposition the quadrant support and the extractor.

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