13H38 / 13H60

Drilling depth	Small	Big	Remarks
1,0	0,05 ml	0,1 ml	Kompakta

12H38 / 12H60

Drilling depth	Small	Big	Remarks
2,0	0,1 ml	0,2 ml	Kompakta / Spongiosa 3:1

11H38 / 11H60

Drilling depth	Small	Big	Remarks
3,0	0, 15 ml	0,25 ml	Kompakta / Spongiosa 2:1

10H38 / 10H60

Drilling depth	Small	Big	Remarks
4,0	0,2 ml	0,3 ml	Kompakta / Spongiosa 1:1

9H38 / 9H60

Drilling depth	Small	Big	Remarks
5,0	0,25 ml	0,35 ml	Kompakta / Spongiosa 1:2

8H38 / 8H60

Drilling depth	Small	Big	Remarks
6,0	0,3 ml	0,4 ml	Kompakta / Spongiosa 1:3

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- Harvesting a pre-defined volume of autologous bone
- Harvesting fine bone chips from cortical bone without the use of a bone mill
- Safe, defined and simple access to all intraoral donor sites with extremely low donor-site morbidity
- Minimally invasive system which is easy to learn and handle
- Time-saving and cost-cutting procedure
- Suitable for preparing defined mixing ratios of bone substitute material and autogenous bone

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Bone Management® is a registered trademark of the Hager & Meisinger GmbH, Germany

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MEISINGER
GERMANY / USA

Harvest-Control

developed with Dr. Bauer, Prof. Dr. Benner

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Harvest-Control is a minimally invasive, very safe and an extremely easy to learn procedure that is used to harvest autologous bone material from all intraoral donor sites. By selecting the corresponding harvest components, the user can achieve a pre-defined bone volume. The process facilitates the production of fine bone chips from cortical bone without the use of a bone mill. Due to its design and adjustable penetration depth, the system results in a very low degree of morbidity at the respective donor site.

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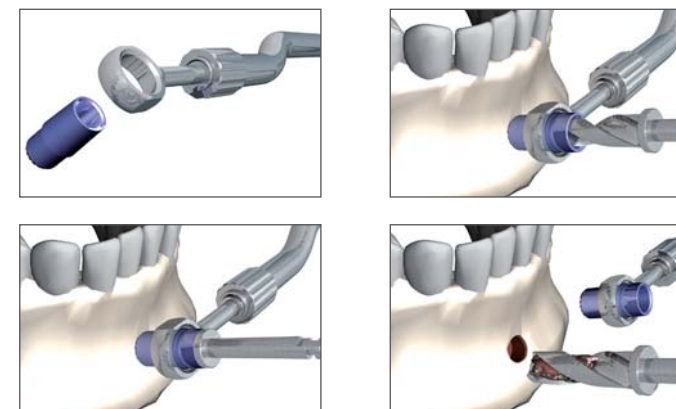
First, a harvesting tool that suits the desired bone volume is chosen with the help of the chart "Bone extraction". The harvesting sleeve is fixed to the drill guide. The respective drill is guided through this sleeve. Now, the harvesting device is placed on the deperiosted bone and the harvesting process starts. Due to the special design, the penetration depth of the drilling system is pre-set. As long as the correct sleeve is chosen, faulty use is impossible.

The system consists of two sets of 6 sleeves whose respective internal diameters measure 3.8 mm and 6.0 mm, in addition to a suitable drill and guiding tool each. The bone penetration depth of the harvesting system varies between 1.0 mm and 6.0 mm.

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¹ 204=RA, 205=RA L ² Largest working part diameter in 1/10 mm Internal diameter

	Bohrhülse Drill sleeves Casquillos portabrocas											
Fig.	13H38	13H60	12H38	12H60	11H38	11H60	10H38	10H60	9H38	9H60	8H38	8H60
Length mm	13,0	13,0	12,0	12,0	11,0	11,0	10,0	10,0	9,0	9,0	8,0	8,0
Drilling depth	1,0	1,0	2,0	2,0	3,0	3,0	4,0	4,0	5,0	5,0	6,0	6,0
Internal diameter	3,8	6,0	3,8	6,0	3,8	6,0	3,8	6,0	3,8	6,0	3,8	6,0
External diameter	5,35	7,55	5,35	7,55	5,35	7,55	5,35	7,55	5,35	7,55	5,35	7,55

Internal diameter External diameter

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General instructions:

- All products delivered are unsterile therefore, before initial and each further potential application to be treated (cleaning / disinfection / sterilisation).

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