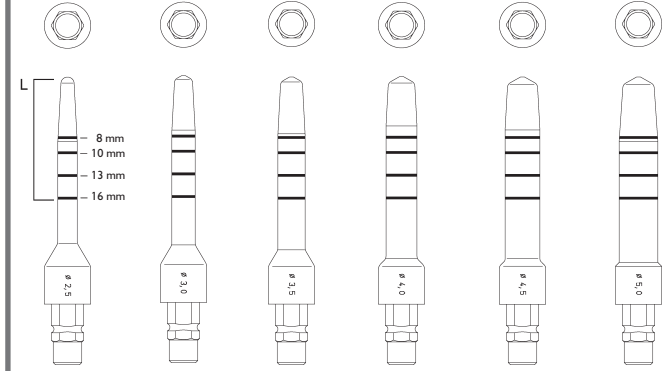
Elevatoren
Elevators
Elevadores


CE 0044

Fig.	SL25	SL30	SL35	SL40	SL45	SL50	SL0
Size ²	025	030	035	040	045	050	
Length mm	16.0	16.0	16.0	16.0	16.0	16.0	100.0


²Largest working part diameter in 1/10 mm

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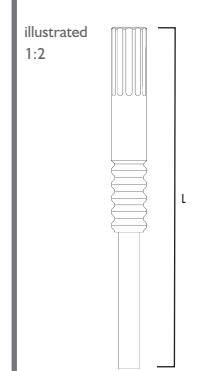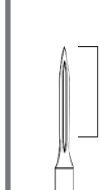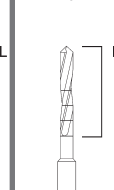
Art.-No.: BSL00

Bone Management® is a registered trademark of Hager & Meisinger GmbH, Germany

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The Bone Management® system Internal-Lift-Control is a system for performing internal sinus lifts where the bone in the implant site is shifted upwards in direction of the maxillary sinus. Summers has developed this technique with transcrestal access to the maxillary sinus. This technique is suitable for augmenting smaller vertical deficits (3 mm - 4 mm) in the sinus with a residual bone height of more than 6 mm (Zitzmann and Schärer 1998, Rosen et al. 1999, Nedir et al. 2006).

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The use of a pre-punch creates a guide channel to prevent any slipping of the twist drill.

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Pre-drilling is then performed with a suitable pilot drill with depth markings. The depth and diameter depend on the available vertical and horizontal bone dimensions, whereby the drill hole should correspond to up to two thirds of vertical bone height (check with x-ray). In as far as no simultaneous horizontal condensation is required, use the expanders to widen the drill hole to the implant diameter according to the manufacturer's protocol.

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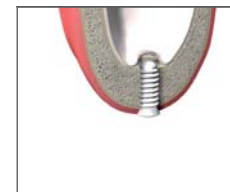
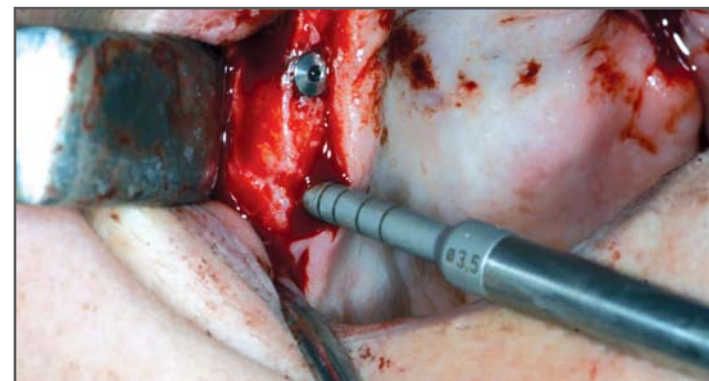
This is followed by further successive preparation of the implant site with increasingly larger elevators. Preparation is performed up to max. 1 - 2 mm below the sinus floor. The sinus floor is now driven vertically into the sinus using the last used elevator including the horizontally condensed bone. Hereby, the mucosa of the maxillary sinus lifts from the floor of the maxillary sinus.

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
In the single-stage procedure, the maxillary sinus is first filled with bone replacement material followed by direct placement of the implant.

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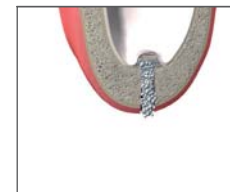In the two-stage procedure, the maxillary sinus is filled with graft material. Following completed new bone formation, the implant can now be placed in a second step.

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- Optimal system for the easy execution of an internal sinus lift
- Controlled minimal invasive procedure
- Vertical and horizontal bone condensing
- Optimal utilization of existing bone substance
- Suitable for preparing the maxillary bone for the placement of all commonly used implants

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General instructions:

- All products delivered are unsterile, therefore, before initial and each further potential application, products should be treated (cleaning / disinfection / sterilization).

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