| CE 0044 | Vorkörner Initial bur Fresa inicial | Diamantierte Sägen Diamond saws Sierras diamantadas | | Kronentrenner Crown cutter Fresa para separar coronas | Osteotomieinstrumente Osteotomy instruments Fresas para osteotomia | | | Ratsche Ratchet Carraca | Eindrehhilfen Jollying aids Soporte para enroscarse | |
|--------------------|---|---|-------|---|--|--------|------|-------------------------------|---|-------|
| | Fig. | 186RF | 231DC | 231DC | HM31C | HM331L | 859 | 859L | CARA1 | CA1RB |
| Shank ¹ | 204 | 204 | 204 | 204 | 205 | 204 | 204 | | | |
| Size ² | 018 | 070 | 130 | 012 | 010 | 018 | 010 | | | |
| Length mm | 12,0 | 0,3 | 0,3 | 4,1 | 5,5 | 10,0 | 12,0 | | | |

¹ 204=RA, 205=RA L ² Largest working part diameter in 1/10 mm

| Fig. | Pilotbohrer Pilot burs Fresas piloto | | Erweiterer Expansion burs Ensanchadores | |
|--------------------|--|-------|---|-------|
| | A1001 | D1001 | A1003 | D1003 |
| Shank ¹ | 206 | 206 | 206 | 206 |
| Size ² | 010 | 018 | 023 | 030 |
| Length mm | 15,0 | 15,0 | 15,0 | 15,0 |
| ▲ | | | 1,03 | 1,75 |
| ◻ | | | 2,30 | 3,00 |

¹ 206=RA XL ² Largest working part diameter in 1/10 mm

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- Minimally invasive alternative to osteotomes
- Controlled dilatation of horizontally resorbed bone
- Gentle densification of cancellous bone
- Suitable for bone resorption classes 1-3
- Provides high primary implant stability through bone condensing
- Suitable for preliminary implant cavity preparation for all current dental implant systems
- Optimal in combination with Horizontal-Control

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Bone Management® is a registered trademark of the Hager & Meisinger GmbH, Germany

Split-Control 15 mm

Bone Spreading and Condensing System

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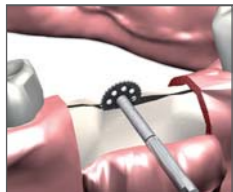
Split-Control is the minimally invasive alternative to osteotomes. Bone Spreading and Bone Condensing with special screw like instruments (spreaders) achieve a controlled and standardised dilatation of horizontally resorbed bone and a gentle densification of cancellous bone. The alveolar ridge can be optimally prepared for the subsequent process of implant insertion while maintaining the existing bone substance without complicated horizontal or vertical bone grafting.

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
To promote Bone Spreading a crestal relief cut with the osteotomy disk (between multi drilling) is implemented.

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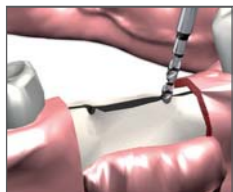
When Bone Spreading the initial bur prevents the pilot bur from slipping. Subsequently the pilot bur produces a subdimensional bone cavity (pay attention to depth of insertion).

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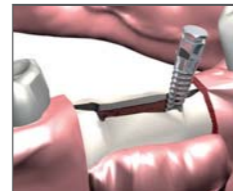The successive deployment of a series of spreaders with increasing diameters leads to Bone Spreading and condensing (order of diameter and codification of the instruments must be respected). With the help of the ratchet the spreaders may be rotated in cautiously. In bone density of 1–2 the drilling before deploying the spreader is prepared with the corresponding D1, D2 expansion bur.

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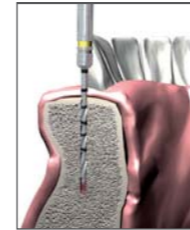
By having widened the implant cavity in this manner it is now possible to place the suitable implant. Furthermore increased bone rigidity, achieved by bone condensation, may lead to greater primary stability.

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
In the case of Bone Condensing first the pilot bur produces a subdimensional bone cavity (take notice of diameter and depth of the drilling). In a narrow alveolar ridge

situation the initial bur prevents the pilot bur from slipping.

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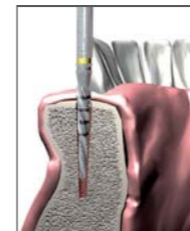A sub-dimensional bone cavity for bone densities D3/D4 is then created using the conical expansion burs.

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For Bone Condensing purposes of implant cavities of more than 2.7 mm diameter, use the expansion bur A1 and the spreader A1. For implant cavities exceeding 3.3 mm diameter, the expansion bur D1 and the spreader D1 must be used. After placement of the primary spreader (A1 or D1), the following spreaders in ascending order can be used without further drilling with pilot drills. In non-atrophied bone, the use of the spreaders leads to condensation of the bone and horizontally resorbed areas of the alveolar ridge are expanded in addition. With the help of the ratchet, the spreaders may be screwed in cautiously.

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The condensed and spread bone may now provide greater primary stability once an implant has been placed.

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General instructions:

- All products delivered are unsterile therefore, before initial and each further potential application to be treated (cleaning / disinfection / sterilisation).

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