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- Secure and easy transplantation of autogenous bone cylinders with a set of precise tools
- Optimally suitable for horizontal and vertical augmentative grafts
- Precise and predictable application due to depth marks
- Suitable for preparing the bone for subsequent insertion of all standard forms of intraoral dental implant fixtures
- Simple, safe transplant of bone cylinders for the restorative construction of a vertically atrophied jawbone using intelligently coordinated instruments
- Prepare a fresh flat contact surface and a central insertion hole for the osteosynthesis screw in a single operation
- Bone cylinder is securely located with no gap
- Optimum site conditions for secure healing
- Considerably reduced operation time due to systematic procedure
- Optimum combination with Bone Management® set Screw System

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Radfräser
Wheel cutters
Fresas ruedas

CE 0044

Fig.	T229L	T229L	T229L	T229L	T229L	TC084	TC084	TC084	TC084	TC084
Shank ¹	205	205	205	205	205	204	204	204	204	204
Size ²	040	050	060	070	080	004	005	006	007	008
Length mm	14,0	14,0	14,0	14,0	14,0	4,0	5,0	6,0	7,0	8,0
Internal diameter	4,0	5,0	6,0	7,0	8,0					
External diameter	5,0	6,0	7,0	8,0	9,0					

¹204=RA, 205=RA L ²Largest working part diameter in 1/10 mm

Lagerfräser
Ablative burs
Fresas ablativas

Diamantierte Sägen
Diamond saws
Sierras diamantadas

Bohrer Verlängerung
Drill extension
Extensión de frasa

Schraubendreher
Screw driver
Atornillador

Fig.	TC21X	TC21X	TC21X	TC21X	TC21X	231DC	231DC	231DC	BV028	SW150
Shank ¹	205	205	205	205	205	204	204	204		
Size ²	040	050	060	070	080	070	100	130		
Length mm	14,0	14,0	14,0	14,0	14,0	0,3	0,3	0,3	28,0	

¹204=RA, 205=RA L ²Largest working part diameter in 1/10 mm

Osteotomieinstrumente
Osteotomy instruments
Fresas para osteotomia

Kronentrenner
Crown cutter
Fresa para separar coronas

Fig.	HM331L	859L	859	HM 31C
Shank ¹	205	204	204	204
Size ²	010	010	018	012
Length mm	5,5	12,0	10,0	4,1

¹204=RA, 205=RA L ²Largest working part diameter in 1/10 mm

Vorkörner
Initial bur
Fresa inicial

Trepan
Trepine
Trépano

Lagerfräser
Ablative bur
Fresa ablativa

Fig.	229VT	229	207RF
Shank ¹	205	205	205
Size ²	075	075	075
Length 1 (mm)	10,0	10,0	3,2
Length 2 (mm)	2,2		3,0
Internal diameter	7,5	7,5	
External diameter	8,5	8,5	7,5

¹205=RA L ²Largest working part diameter in 1/10 mm

Bone Management® is a registered trademark of the Hager & Meisinger GmbH, Germany

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This Bone Replacing System, suitable for all current dental implant systems, permits precise and standardised transplantation of autogenous bone cylinders. Different types of bone burs are available according to three groups of diameters. Within each group the instruments are adapted to each other so that the outer diameter of the ablative bur or the diameter of the wheel cutter correlates with the inner diameter of the trephine. A press fit is often produced between the cylindrical bone autograft and the prepared receiving site, which can be strengthened with a fixation screw (sold separately). Such precise fitting of autografts lead to accelerated bone revitalisation and wound-healing, thus creating a perfect site for dental implantation within 3 to 4 months.

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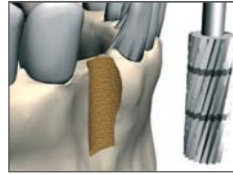
In cases of severe horizontal alveolar ridge resorption (where even bone-spreading techniques cannot be deployed) horizontal augmentative grafts are required for compensation purposes.

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Initially the ablative bur (ensure diameter) is chosen and deployed to drill a standardised cavity against the horizontally resorbed alveolar ridge according to the diameter of the subsequently inserted implant. The surgical technique and the properties of the ablative bone bur lead to a hyperaemic environment, which accelerates the wound healing in post transplantation.

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With the appropriate trephine, corresponding to the previously used ablative bur, a bone cylinder congruent to the diameter of the recipient region is extracted e.g. from the retro-molar region of the mandible. If necessary, the osteotomy disk or diamond can be used to extract the bone cylinder from the donor region.

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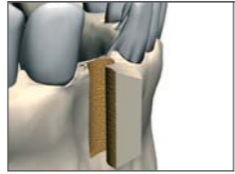
The donor region may be filled with collagen or other grafting material. The intervention is terminated by a current gingival suture.

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The extracted bone cylinder is placed into the recipient bone site to compensate the horizontal bone deficit. A satisfying press fit may render additional fixation unnecessary.

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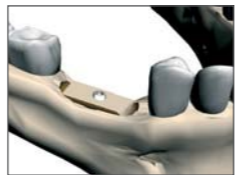
If a certain mobility of the autograft in the cavity remains, a fixation screw from the Screw System (Art.-No. BSR00) which optimally corresponds with Transfer-Control, can be implemented.

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In cases of severe vertical alveolar ridge resorption, **vertical augmentative grafts** are required for compensation. In this case the appropriate wheel-cutter instead of an ablative bur creates a congruent cavity in the crestal alveolar ridge. Thereby one should take note of the correlating trephines to be used. Prepare an autogenous bone graft in the same manner as described for horizontal defects.

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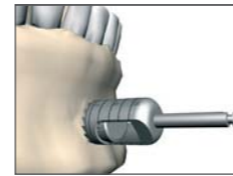
The most important condition for a safe healing of an osseous graft is a congruent and fresh contact surface of the implant area. The Transfer-Ring-Control system allows this precondition for the healing of vertical plastic supports in a simple and controlled manner – while at the same time reducing operation time considerably. The Bone Management® set Screw System (order no. BSR00) is the ideal complement to the Transfer-Ring-Control.

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
The initial phase requires the donor region to be marked. The hole thus created is used later to ensure perfect centring of the osteosynthesis screw in the implant site, at the same time, the surface of the intended bone cylinder is ground flat thus creating a radial groove.

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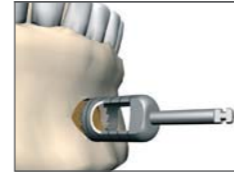Next, the trephine is used to cut a bone cylinder of the correct diameter from the donor site. Depending on the vertical requirements at the site, this cylinder can have a height of 4 mm to 7 mm.

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The bone cylinder is carefully removed from the donor site which can then be filled with collagen.

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If the surface of the cylinder is still uneven, it can be ground flat in a simple and controlled manner using the ablative drill. Similarly, the central guide hole for the osteosynthesis screw can be drilled.

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Next, the ablative bur is used to form a standardised bone site with a flat, fresh contact area. The bone cylinder can then be inserted perfectly into the hole. Since the instruments used are all designed to complement each other, the transplant will be a perfect, gap-free fit.

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Once the bone cylinder has been inserted in the site, it is secured using the osteosynthesis screw (can be ideally combined with the screws in the Bone Management® set Screw System).

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If required, splinters on the upper surface of the bone cylinder can be removed using the diamond grinder.

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General instructions:
• All products delivered are unsterile therefore, before initial and each further potential application to be treated (cleaning / disinfection / sterilisation).

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