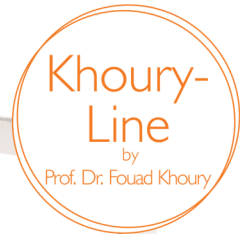
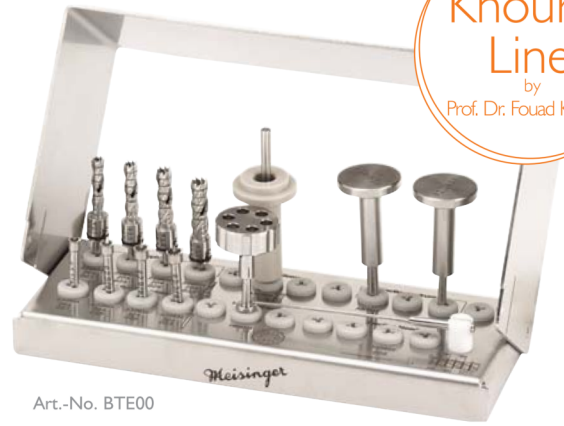
Trephine Ejection Kit

Trephine System for Extraction of Cylindrical Bone Grafts | developed by Prof. Dr. Fouad Khoury

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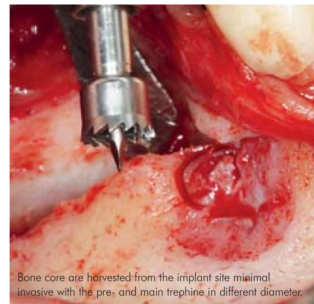
Art.-No. BTE00

The Trephine Ejection Kit was developed together with Prof. Dr. Fouad Khoury. It contains several unique ideas which turn this kit into something special: Two-part, internally cooled trepanes allow for a safe and minimal invasive extraction of local bone cylinders. The inner cooling helps to prevent a damage of the bone tissue due to overheating, while the removable trephine working parts allow for an especially easy sampling of the cut out bone cylinders. Either with the aid of the ejection needle or the ejection instruments, which perfectly match with the diameter of the trephine working parts, the bone cylinders can be pushed out of the trepanes. Furthermore, the included pre-drilling trepanes allow for a safe application of the trepanes. Due to the two-part trepanes, the variety of different diameters and the optimal matching of the instruments, the Trephine Ejection Kit offers highest efficiency, flexibility and precision to the user for the extraction of bone cylinders.


Bone core are harvested from the implant site minimal invasive with the pre- and main trephine in different diameter.


Bone core augmentation stabilized with micro-screws

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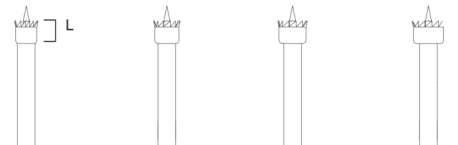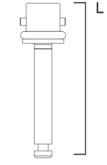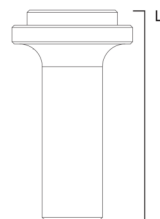
AT A GLANCE

- Two-part trephines for a safe and minimal invasive extraction of bone cylinders
- Protection against overheating due to internally cooled trephines
- Innovative application
- Standardized diameters and precise laser markings of the trephines allow for a predictable extraction of defined bone cylinders
- Perfectly matched instruments care for highest precision and flexibility

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© Prof. Dr. Fouad Khoury